




## Cartridge and infusion set change in combination with mylife CamAPS FX

### General information regarding the cartridge and infusion set change in combination with mylife CamAPS FX

- Auto mode should be left ON during an infusion set and/or cartridge change. The more time spent in Auto mode, the better the learning of the algorithm will be.
- When taking off the mylife YpsoPump for more than 15 minutes, for example if you take a bath while changing the infusion set, please put the mylife YpsoPump into stop mode.
- If you put your mylife YpsoPump into stop mode, remember to switch the mylife YpsoPump back into run mode. Otherwise, the mylife YpsoPump will alert you if the insulin pump has been in stop mode for more than one hour. The mylife CamAPS FX app will alert every 30 minutes when the pump is suspended.
- The screen background turns orange (attempting) when you put the mylife YpsoPump into stop mode. If you switch back to run mode, the app's screen background turns green (Auto mode on).

1


### Put the mylife YpsoPump into stop mode

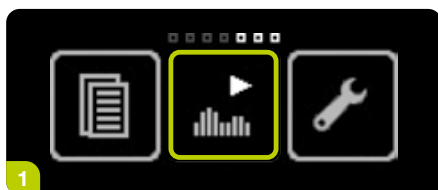

1. Swipe left to open the main menu and unlock the mylife YpsoPump. Swipe to the left and tap the “Run mode/Switch to stop mode” icon. If there is a running extended bolus or a temporary basal rate running on the home screen, you can still swipe to the left.



2. Confirm by tapping ☒.

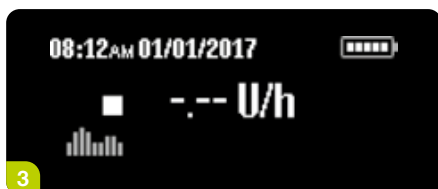

3. The status screen appears. Insulin delivery is stopped immediately. This is indicated on the screen.



## 2

## Changing the infusion set and cartridge

Now perform the cartridge and/or infusion set change as described in chapter 5 of the mylife YpsoPump Instruction for Use.

You can download the Instruction for Use here:

<https://pharmacodiabetes.co.nz/products/mylife-ypsopump-and-consumables/mylife-ypsopump-starter-pack/>

Or you can watch the mylife YpsoPump handling videos:

<https://pharmacodiabetes.co.nz/mylife-ypsopump/ypsopump-handling-videos/>

## 3

## Set the mylife YpsoPump and the mylife CamAPS FX app into operation again

Put the pump back into run mode as described in chapter 2.4 of the mylife YpsoPump Instruction for Use. Auto mode will resume automatically within a few minutes and the app's screen background will change from orange to green.



mylife YpsoPump and CamAPS FX are class IIb devices. Only rapid-acting insulin at a concentration of 100 U/ml (insulin analogue) may be used with the mylife YpsoPump system. Always read the label and use only as directed. Consult your healthcare professional to see if these products are right for you.

**Disclaimer:**

This is a medical device. Use it according to the instructions for use or label.

The product images are for illustrative purposes only. // mylife and YpsoPump are registered trademarks of Ypsomed AG. // CamAPS is a registered trademark of CamDiab Ltd. // The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc., and any use of such marks by Ypsomed is under license. // Other trademarks and trade names are those of their respective owners.

 For safety information on mentioned products, see Instructions for Use