

CareSens N Blood Glucose Meter

Data Port


Used to transfer data from the meter to a computer with the cable available from your authorised i-SENS representative

↑ Button

Selects or changes information

S Button

Turns the meter on/off, confirms menu selections, and changes information

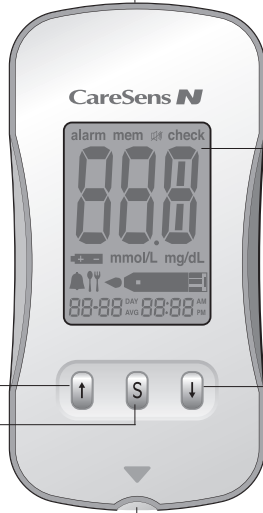

Test Strip Port
Insert test strip here

Display

Shows results, messages

↓ Button

Selects or changes information

Note:

- The cable for data management software can be ordered separately. Please contact your authorised i-SENS sales representative.
- The unit of measurement is fixed in mmol/L and it cannot be changed by the user.

CareSens N Blood Glucose Meter Display

mem

appears when test results stored in the memory are displayed

alarm

appears when the time alarm has been set

Battery symbol

indicates meter battery is running low and needs to be replaced

Post-meal test flag

appears during post-meal testing and when post-meal test results are displayed

Alarm symbol

appears when the post-meal alarm has been set

mmol/L

unit for measuring blood glucose

Sound symbol

appears only when the sound is set to OFF

check

appears when the control solution test results are saved or displayed

Test results

test results displaying panel

Decimal point


appears when the blood glucose measuring unit is set to mmol/L

mg/dL

unit for measuring blood glucose

Blood insertion symbol

indicates meter is ready for the application of a drop of blood or control solution

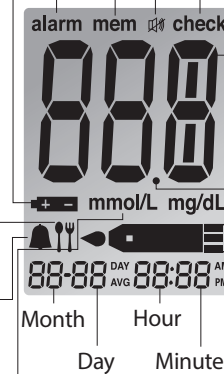

Note: It is recommended to check if the display screen on the meter matches the illustration above every time the meter turns on. Do not use the meter if the display screen does not exactly match the illustration as the meter may show incorrect results.