

## Inserting or Replacing the Batteries

The CareSens N POP meter uses two 3.0V lithium batteries. Before using the meter, check the battery compartment and insert batteries if empty.

When the **+ -** symbol appears on the display while the meter is in use, the batteries should be replaced as soon as possible. The test results may not be saved if the batteries run out completely.



### Step 1

Make sure the meter is turned off. Press down and slide off the battery compartment cover.




### Step 2

Remove the old batteries one by one by lifting with the index finger and pulling it out with your thumb and index fingers as shown in the diagram on the right. Insert two new batteries with the + side facing up and make sure the batteries are inserted firmly.

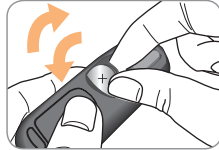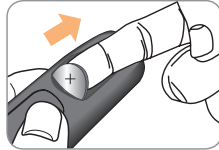

### Step 3

Place the cover on the battery compartment. Push it down until you hear the tab click into place.

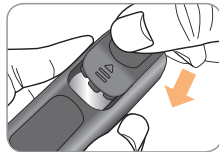

**Note:** Removing the meter batteries will not affect your stored results. However you may need to reset your meter settings. See page 14-19.

## Caring for Your System

Use a soft cloth or tissue to wipe the meter exterior.

If necessary, the soft cloth or tissue might be dipped in a small amount of alcohol.

Do not use organic solvents such as benzene, or acetone, or household and industrial cleaners that may cause irreparable damage to the meter.

### Caution:

- Do not expose the meter to direct sunlight, heat, or excessive humidity.
- Do not let dirt, dust, blood, or water enter into the meter's test strip port.
- Do not drop the meter or submit it to strong shocks.
- Do not try to fix or alter the meter in any way.
- Keep the meter away from strong electromagnetic fields such as cell phones and microwave ovens.
- CareSens N POP meter should be used only with the CareSens N test strips.
- Store all the meter components in the carrying case to prevent loss and help keep the meter clean.